


KWICKSCREEN

DUO USER MANUAL

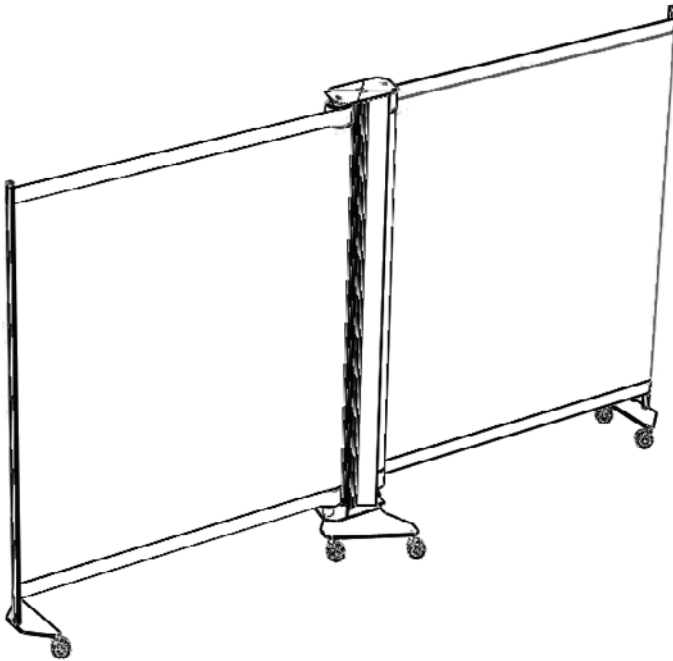

We transform spaces by making them flexible, enriching their beauty, functionality and efficiency with portable, retractable, printed partitions.

Please read all instructions in this user guide prior to installation and operation of your KwickScreen.

For personal and operational safety use only parts approved by KwickScreen.

Be gentle when pulling the KwickScreen out for use.

Before installation ensure all parts are present, complete and without defects.

KwickScreens must be allowed to find their "natural position". If you find the screen difficult to push in or pull out, or you hear a crumpling sound, then reverse your action (push if pulling and pull if pushing).

If you believe parts to be missing please contact us.

Packaging material should be disposed of in accordance with local rules and regulations.

CONTENTS

3/ welcome

4/ view & dimensions

5/ replacing or changing inner panel

7/ unit assembly guide

14/ cleaning

If you have any questions please contact us at:
www.kwickscreen.com / info@kwickscreen.com
+44 (0) 20 8452 5975

Or visit us at our showroom and offices at Unit 3, Phoenix Trading Estate, Bilton Road, London.
UB6 7DZ

Dear valued customer,

Thank you for choosing to purchase from KwickScreen.

I hope that your new KwickScreen will help you make better use of your spaces and if you have chosen a printed screen, that you enjoy the benefits which the images can bring to your environment.

I am passionate about using technology to improve environments. As a designer and engineer I invented the KwickScreen to help solve space issues. For a room to be used efficiently and appropriately it needs to be flexible both spatially and visually. Every multi-use room, from a hospital ward, to a gym, a private meeting room or an airport has problems and limitations that can be solved by space saving portable screening systems.

Over the last ten years the product has developed significantly, winning a number of design and innovation awards. We now have an international supply chain with distribution in Europe, North Asia, America and the Middle East.

Our customers and those who use and interact with our screens remain our top priority and we are always striving to both improve the product and our service to you. This is our passion and that is why we assemble and quality check every product ourselves locally in England.

We hope that you have received the product in a perfect condition, but if for any reason you are unhappy or have a query please let us know and we will do our best to help. Each unit and screen has a unique code making it possible for us to trace all relevant information about the product. (The unit number Uxxx is written on the back of the unit on the silver sticker and the screen number Sxxx is printed on the inside of the front of the screen.)

KwickScreens are used in all manner of places, so if you can we would love to receive pictures and feedback from you about using your KwickScreen whether it's a query, comment or suggestion. Please find us on social media @KwickScreen or alternatively get in touch via our website www.kwickscreen.com.

I wish you many years of use from your KwickScreen and look forward to hearing from you.

Michael Korn
Inventor and Co-Founder of KwickScreen





VIEW & DIMENSIONS

Fully Assembled Weight: 24 kg

Bases Weight: 5 kg

Unit Weight: 19 kg

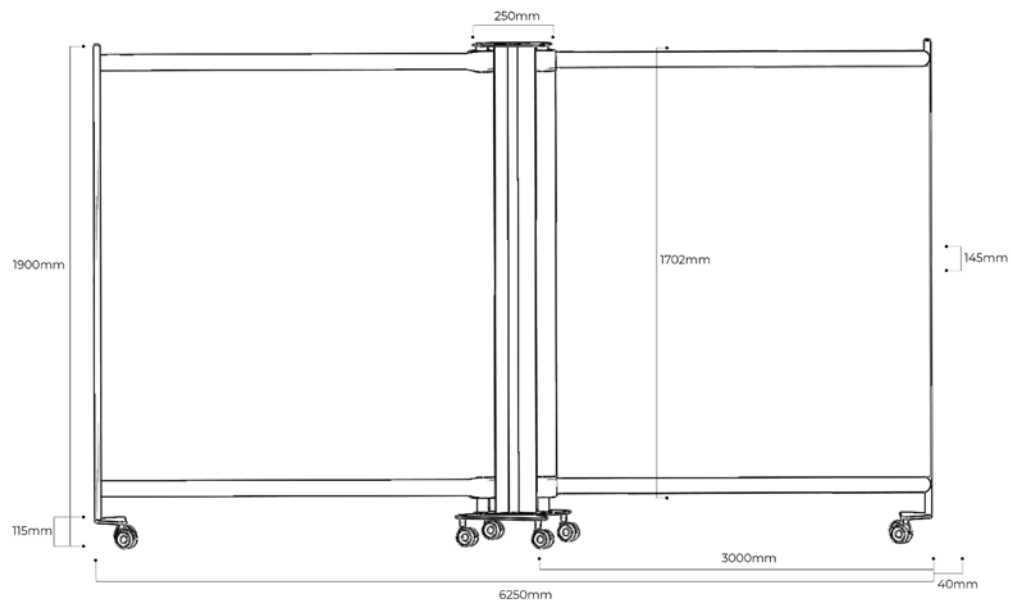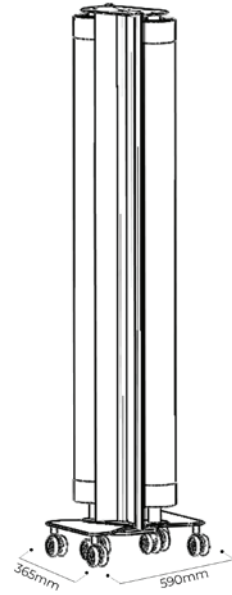


REPLACING OR CHANGING THE INNER PANEL (2 person job)

The images show a KwickScreen Pro, however the process is the same for the Duo. Repeat for both inner screens.


1

Extend unit fully (unit shown with inner screen panel removed) by applying all four brakes on the main body and pulling out the handle.

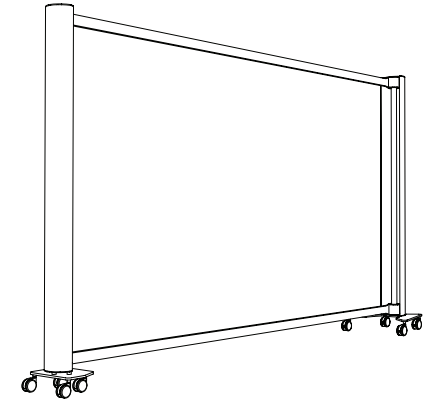


2

To insert the inner panel keep the panel rolled up and attach the zip in the top left corner. To get started one person should roll whilst the other starts the zip.

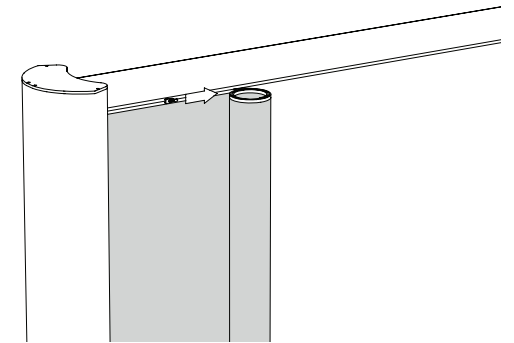


3

Support the rolled panel in one hand, slowly zip along the top of the panel as you unroll the panel.

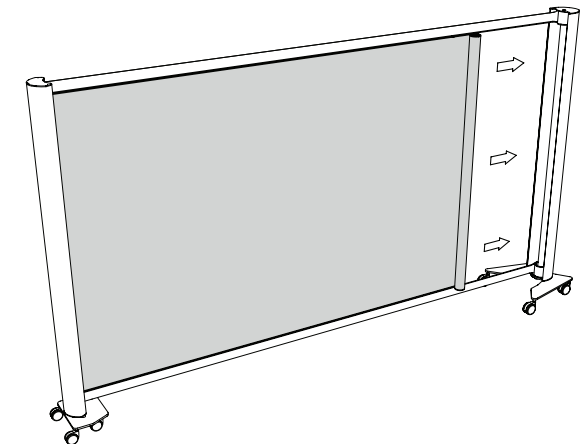


4

Leave 500mm unzipped and thread the panel end around the plastic core. Carefully zip up the internal zip on the panel to secure it to the rotor and then zip the top zip to the end.

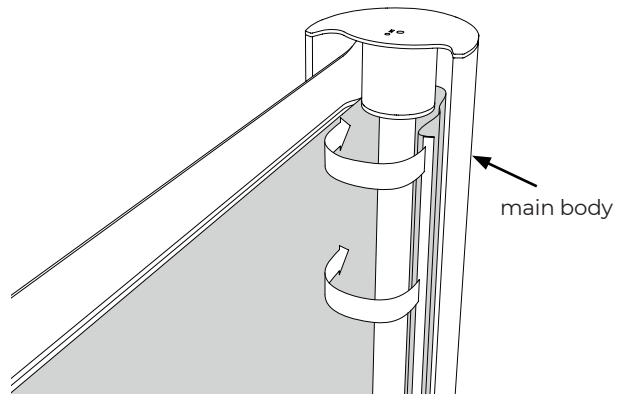


5

Align the magnetic strip on the front handle

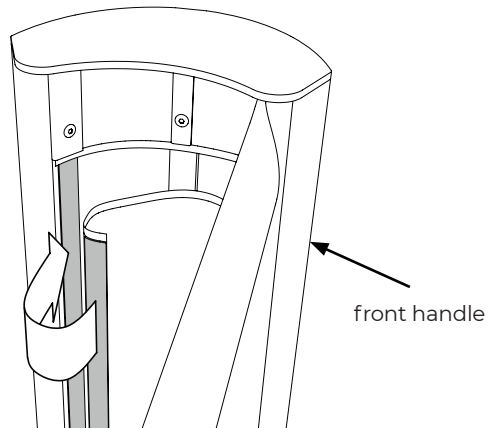

6


Close the bottom zip

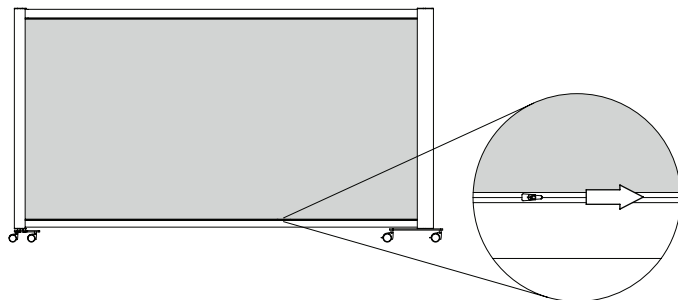


ASSEMBLY GUIDE

KWICKSCREEN DUO UNIT


KwickScreen Duo main body with two inner screens

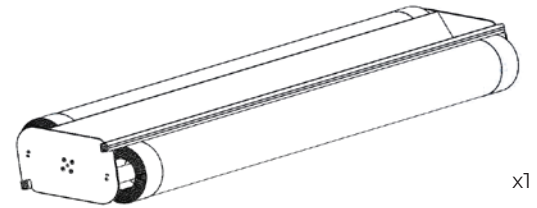

main body base

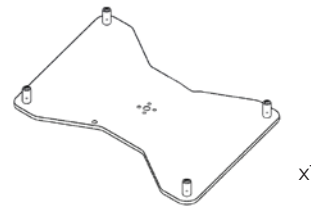


front handle bases

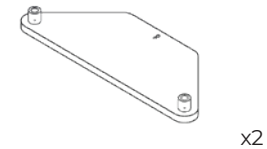


braked castors for main body




unbraked castors for front handles

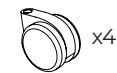


castor pin

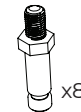

M6 cap head screw

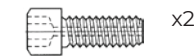

M5 cap head screw

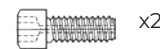


4mm allen key



13mm spanner



3mm allen key

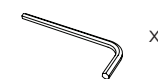

Please read the following instructions.

This is a two person job.

Please ensure all screws are tight after assembly.

If you have any questions please contact us at:

www.kwickscreen.com



call us on:

0208 452 5975


or email us:

info@kwickscreen.com

Attach wheels to all three bases.

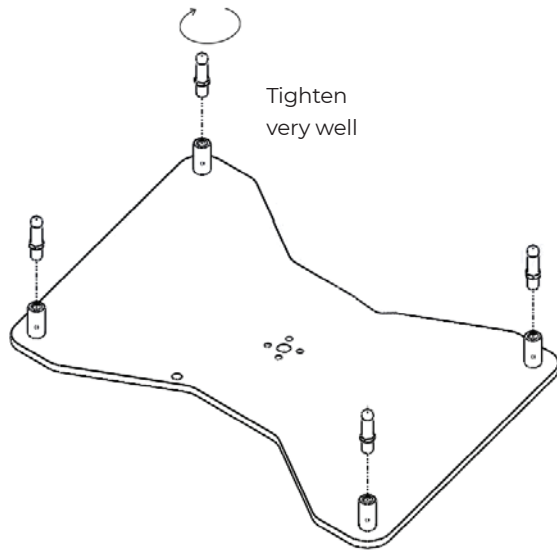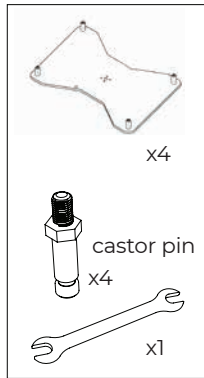


Step 1

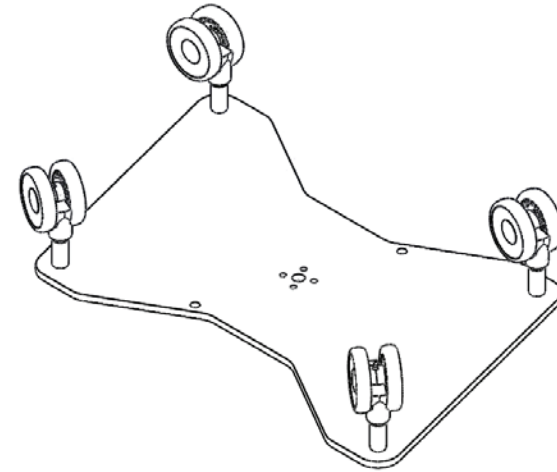


The Duo has two handle bases each with two wheels.


Push on the wheels **with brakes** onto main body base

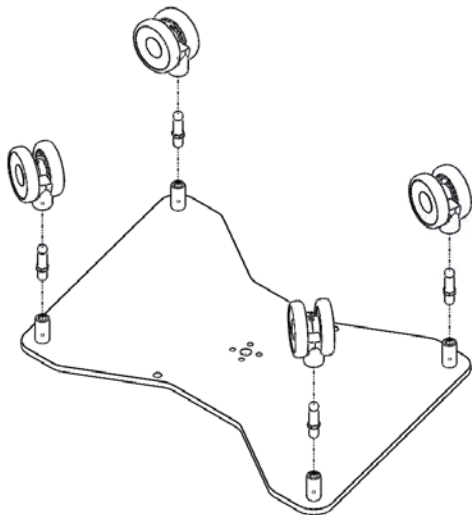



Step 2

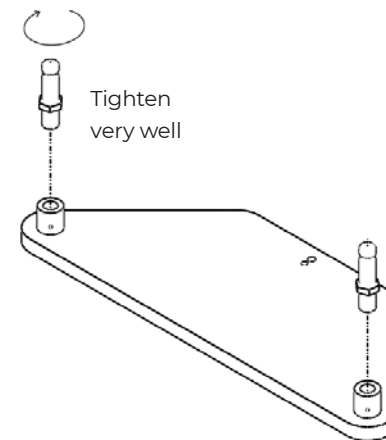

Step 3

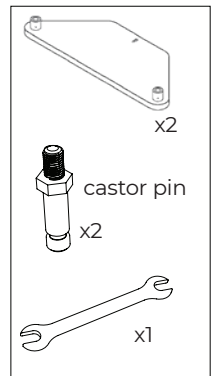


Push on the wheels **without brakes** onto handle base, repeat for both bases.

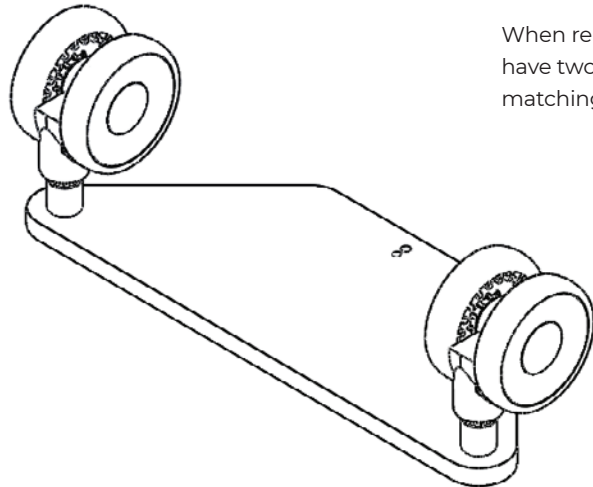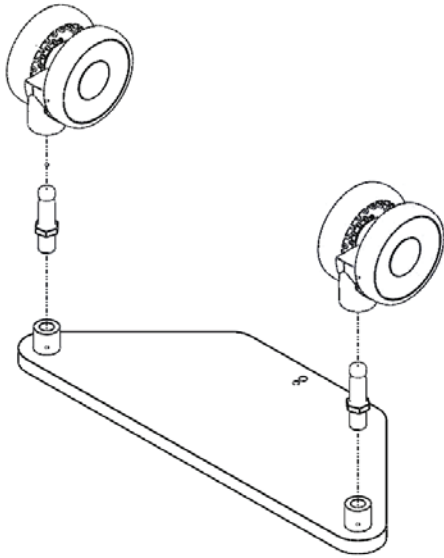



When repeated you will have two handles bases matching this


Step 4

Make sure the flat side is against the body



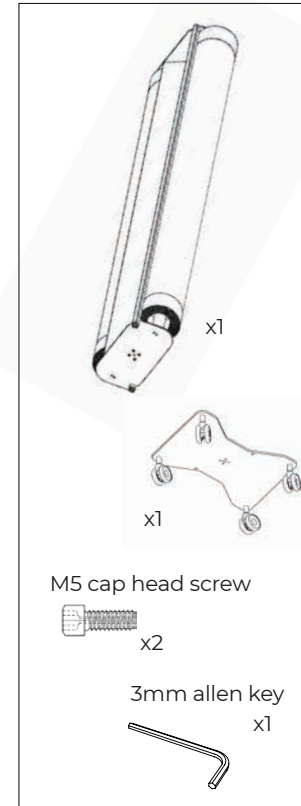


Step 5

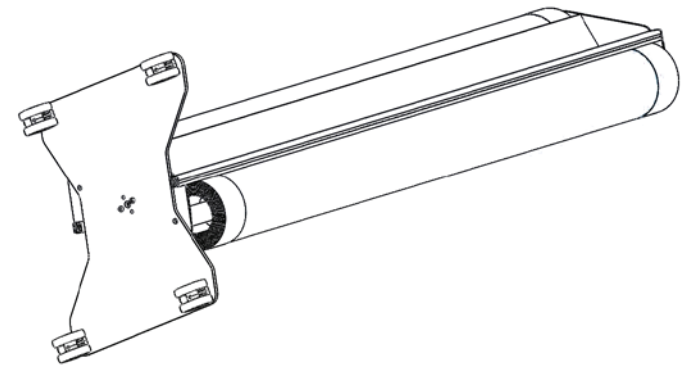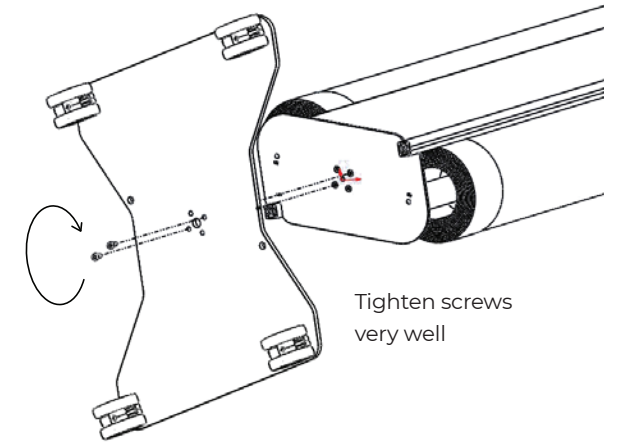

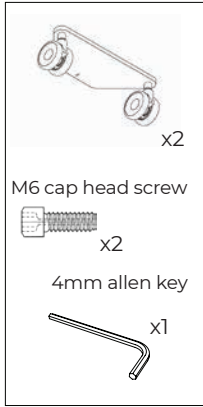

One screw per base

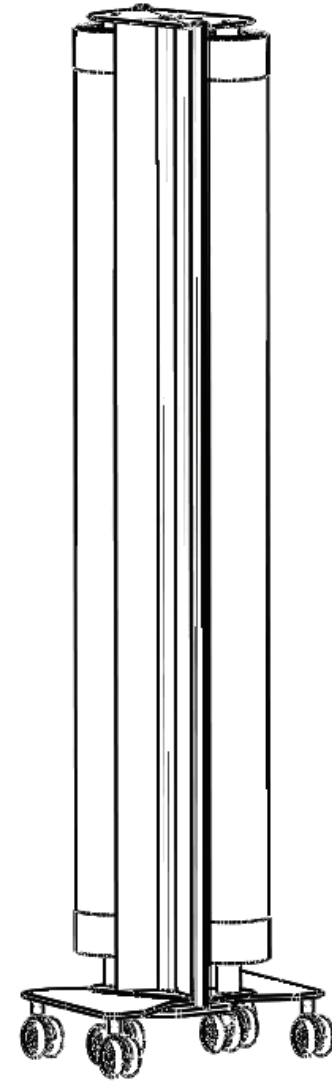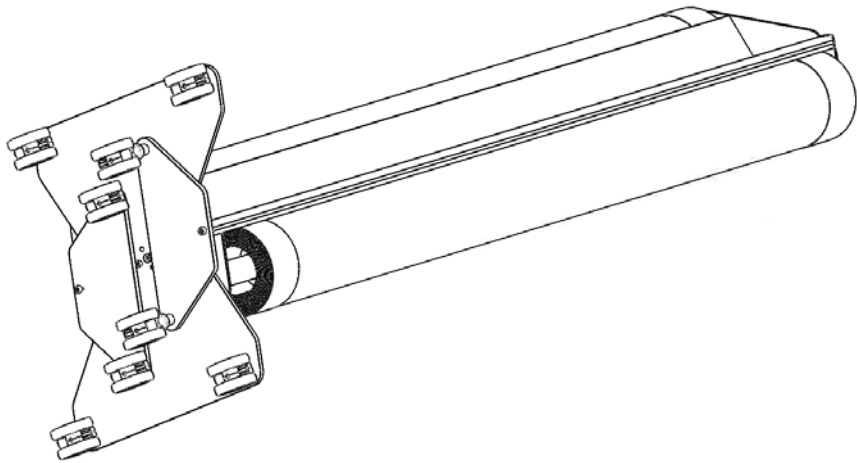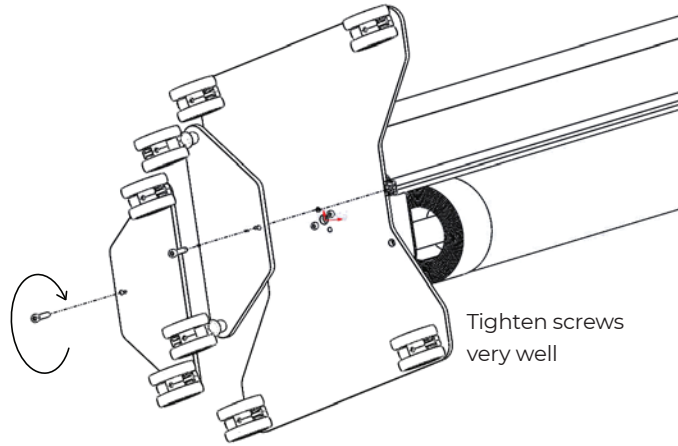

CLEANING

KwickScreens have been designed in line with infection control performance specifications. It's smooth surfaces are easy to clean and designed to withstand regular cleaning. The body is made of anodized aluminium, other metal parts are powder coated steel. The screen panel is made with polyester film.

NORMAL CLEANING

Surfaces can be cleaned using most healthcare cleaning solutions including quaternary, bleach solutions or sporicidal and virucidal disinfectants.

Hypochlorite may also be used on the screen material. Hypochlorite can be applied at 1000-10000 ppm or a 60% alcohol solution.

STUBBORN STAINS

If normal cleaning does not remove all stains the following cleaning agents may be used at room temperature:

- Methyl butyl alcohol
- Naphtha VM&P
- Ethyl alcohol
- Isopropyl alcohol

Care should be taken with printed panels when using alcohol based agents.

DEEP CLEANING

Deep cleaning is recommended every 3-4 months. To deep clean, fully extend the screen and clean inside the body.



WHEN CLEANING THE KWICKSCREEN DO NOT USE:

- Wax polishes or solvents
- Products with an abrasive surface as steel wool
- Detergents which can form lime or synthetic detergents
- Alkalis (over pH10)
- Solvents
- Gasoline
- Greases or oils
- Abrasive cloths or kitchen scouring compounds to clean the printed screen panel
- Any sharp instruments or razor blades to scrape the surface

Do not clean in direct sunlight or in elevated temperatures.

AFTER CLEANING, LEAVE THE SCREEN EXTENDED TO DRY
DO NOT RETRACT WHILST THE SCREEN IS WET

OPERATION

GENERAL INSTRUCTIONS

KwickScreens are designed for use on smooth, horizontal surfaces. Extra care should be taken when using on other surfaces.

When moving the KwickScreen Duo check all four brakes have been released. The unit should then be moved using the base handles.

Push and pull the front handles gently during operation. If they need to be forced please contact us.

If there is resistance or you hear a crumpling sound then stop, pull out and push in again gently.

Make sure the front handles are vertical and that its wheels are all in contact with the floor.

If leaving the KwickScreen on an incline please ensure all brakes are engaged.

Do not run when using a KwickScreen.

DAMAGE PROCEDURE

If you do experience any problems or damage with your screen, please email us (with images if applicable) to info@kwickscreen.com.

KWICKSCREEN

If you have any questions please contact us at:
www.kwickscreen.com / info@kwickscreen.com

You can call us on: +44 20 8452 5975

or visit us at:
Unit 3, Phoenix Trading Estate, Bilton Road,
London. UB6 7DZ