



MED SYSTEMS


2631 ARIANE DRIVE
SAN DIEGO, CA 92117

Tel: 800-345-9061 FAX: 858-483-9827

SALES AND SERVICE
O.E. MEYER CO.
MEDICAL PRODUCT DIVISION

3303 TIFFIN AVE. • P.O. BOX 479
SANDUSKY, OHIO 44870

PHONE (419) 625-3054
1-800-221-9696

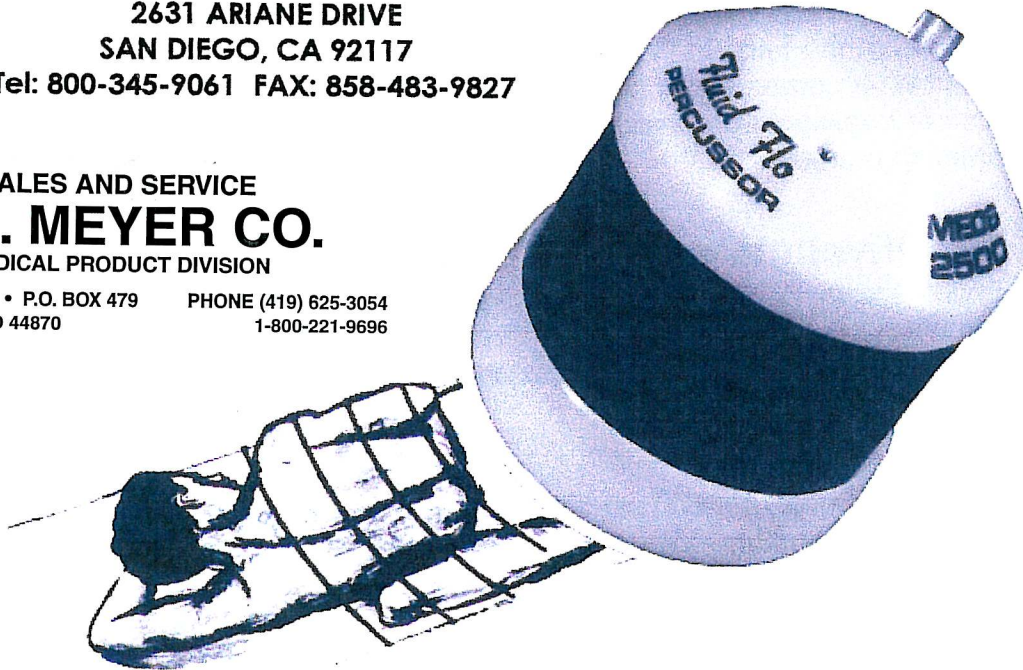

GO FOR THE PULSE !!

MODEL 2500 Fluid Flo Percussor

CONGRATULATIONS ON SELECTING THE BEST HANDHELD PNEUMATIC PERCUSSOR.

THE FLUID FLO PERCUSSOR ...

- DELIVERS DEEP, SMOOTH, HARMONIC PULSATIONS THAT SUCCESSFULLY CLEAR THE PATIENT'S LUNGS.**
- IS LIGHTWEIGHT (ONLY 1.5 lbs.) AND EASY TO USE.**
- MIMICS HAND PERCUSSION WITHOUT FATIGUING THE PATIENT OR CAREGIVER.**
- IS RENOWNED FOR ITS RELIABILITY.**

INDICATIONS FOR CHEST PHYSIOTHERAPY (CPT)...

- COPD, CHRONIC OBSTRUCTIVE PULMONARY DISEASE**
- CYSTIC FIBROSIS**
- MUSCULAR DYSTROPHY**
- RESTRICTIVE PULMONARY DISEASE**
- ICU- POST-OP SETTING (PNEUMONIA OR ATELECTASIS)**
- CEREBRAL PALSY**
- BRONCHIECTASIS**

Please keep this brochure and accompanying information for the Fluid Flo Percussor.

www.medsystems.com

ABOUT THE FLUID FLO

Since 1976, Med Systems has designed, manufactured and delivered superior quality respiratory products. By using our product, you'll join the thousands of satisfied customers currently benefiting from the Fluid Flo Percussor. Each unit is available with DISS Oxygen, DISS Medical air or an air compressor hose assembly. Additional quick connects or dual outlet "y" adapters are available upon request. Please take a moment to familiarize yourself with the accompanying material.

What is Chest Physiotherapy (CPT)?

Chest Physiotherapy (CPT), also called Chest Physical Therapy, is an airway clearance technique that combines manual or mechanical percussion of the chest wall by a caregiver, strategic positioning of the patient for mucus drainage, cough and breathing techniques.

INSTRUCTIONS

- Connect the hose assembly to a 50 psi gas source @ 2cfm. Always match gas source with manufacturer's hose assembly. Do not attach a flow meter, as it will degrade performance.
- Place the patient in the first drainage position. Use a soft towel between the percussor and the patient.
- To activate the pulse speed, turn the knob counter clockwise. Some initial gas flow is required before the pulse begins. One entire turn may be needed to start. Two turns is full speed.
- To control the pulse intensity, hold the front cover against the patient being careful not to press too firmly.
- Move the patient into a variety of drainage positions. Vary speed and intensity as needed.
- Wipe external surfaces with alcohol after each use for cleaning.

CAUTIONS

- The Fluid Flo Percussor is intended for use by trained personnel only.
- Do not use oil or autoclave.
- Federal law restricts this device to sale by or on the order of a physician.
- Not intended for infants or children less than 50 lbs.

WARRANTY

This Med Systems product is warranted against defective materials or workmanship for one year after date of purchase. This product is intended for use by trained personnel only. Med Systems shall, upon receipt of written claim of defective material or workmanship and upon inspection of parts claimed to be defective, repair, or at its option replace any part which Med Systems determines to be defective. When returning equipment for warranty service, shipping charges must be prepaid. Warranty repairs will be made and returned, shipped at no charge. This warranty does not cover ordinary wear and tear arising from the normal use of the equipment. The forgoing warranty is in lieu of all other warranties whether expressed or implied including, but not limited to, any warranty of merchantability of fitness for a particular purpose shall exist in connection with the sale or use of this product.

SALES AND SERVICE

- Contact us to arrange a free 30-day trial.
- Business hours Monday-Friday 6:00 a.m. to 4:00 p.m. (PST) 800-345-9061.
- References available upon request.